



Storage & Disposal

RDO GOLD Clear and Controllable Decalcifier

STORAGE & DISPOSAL

Directions:

1. Store at room temperature. Keep container closed at all times. Store only in a glass or plastic container. Do not use metal containers, as **RDO GOLD** will can pit or oxidize some metals after long periods of exposure.
2. After long periods of storage, some change of color or an increase of suspended precipitate may occur. These are normal occurrences and do not affect the decalcifying potential of **RDO GOLD**. The precipitate may be allowed to settle or removed by filtration; however, neither action is necessary.
3. **RDO GOLD** is biodegradable as received and may be disposed of down regular city sewer systems with water flush. Dispose of according to federal, state and local regulations.